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SARSTEDT Safety-Multifly® Kanülen / Multifly® Kanülen

DE	Gebrauchshinweis – SARSTEDT Safety-Multifly® Kanülen/Multifly® Kanülen	2 - 4
EN	Instructions for use – SARSTEDT Safety-Multifly® Needles/Multifly® Needles	5 - 8
BG	Инструкции за употреба – SARSTEDT Safety-Multifly® Канюли/Multifly® Канюли	9 - 12
CS	Návod k obsluze – SARSTEDT Jehel Safety-Multifly®/Jehel Multifly®	13 - 16
DA	Brugsanvisning – SARSTEDT Safety-Multifly® Kanyler/Multifly® Kanyler	17 - 20
EL	Οδηγίες χρήσεως – SARSTEDT Βελόνες Safety-Multifly®/Βελόνες Multifly®	21 - 24
ES	Instrucciones de uso – SARSTEDT Aguja de seguridad Multifly®/Agujas Multifly®	25 - 28
ET	Kasutusjuhend – SARSTEDT Safety-Multifly® Kanüülid/Multifly® Kanüülid	29 - 32
FI	Käyttöohje – SARSTEDT Safety-Multifly®-Kanyylit/Multifly®-Kanyylit	33 - 36
FR	Mode d'emploi – SARSTEDT Aiguilles Safety-Multifly®/Multifly®	37 - 40
HR	Uputa za upotrebu – SARSTEDT Kanile Safety-Multifly®/Kanile Multifly®	41 - 44
HU	Használati utasítás – SARSTEDT Safety-Multifly® Kanülök/Multifly® Kanülök	45 - 48
IT	Istruzioni d'uso – SARSTEDT Aghi Safety-Multifly®/Aghi Multifly®	49 - 52
KO	사용 설명서 – SARSTEDT Safety-Multifly® 바늘/Multifly® 바늘	53 - 56
LT	Naudojimo instrukcijos – SARSTEDT „Safety-Multifly®“ Kaniulės/„Multifly®“ Kaniulės	57 - 60
LV	Lietošanas norādes – SARSTEDT Safety-Multifly® Kanīles/Multifly® Kanīles	61 - 64
NL	Gebruiksaanwijzing – SARSTEDT Safety-Multifly®-Naalden/Multifly®-Naalden	65 - 68
NO	Bruksanvisning – SARSTEDT Safety-Multifly®-Kanyler/Multifly®-Kanyler	69 - 72
PL	Instrukcja obsługi – SARSTEDT Igły Safety-Multifly®/Igły Multifly®	73 - 76
PT	Instruções de utilização – SARSTEDT Agulhas Safety-Multifly®/Agulhas Multifly®	77 - 80
RO	Instrucțiuni de utilizare – Canule SARSTEDT Safety-Multifly®/Canule SARSTEDT Multifly®	81 - 84
RU	Инструкция по применению – SARSTEDT Безопасные иглы-бабочки Safety-Multifly®/ иглы-бабочки Multifly®	85 - 88
SK	Návod na Použitie – SARSTEDT Safety-Multifly® Ihly/Multifly® Ihly	89 - 92
SL	Navodila za uporabo – SARSTEDT Kanila Safety-Multifly®/Kanila Multifly®	93 - 96
SV	Brugsanvisning – SARSTEDT Safety-Multifly®-Kanyler/Multifly®-Kanyler	97 - 100
TH	คำแนะนำในการใช้งาน – SARSTEDT หัวเข็ม Safety-Multifly®/หัวเข็ม Multifly®	101 - 104
TR	Kullanım için talimatlar – SARSTEDT Safety-Multifly® İğneler/Multifly® İğneler	105 - 108

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- CLSI* GP41 Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture.
- *CLSI (Clinical and Laboratory Standards Institute)
- WHO guidelines on drawing blood: best practice in phlebotomy
- DIN 13097-4: Hypodermic needles - Part 4: Point geometry, requirements and testing.
- DIN EN ISO 6009: Hypodermic needles for single use - Colour coding for identification.
- DIN EN 10088-1: Stainless steels - Part 1: List of stainless steels.
- DIN EN ISO 9626: Stainless steel needle tubing for the manufacture of medical devices - Requirements and test methods.

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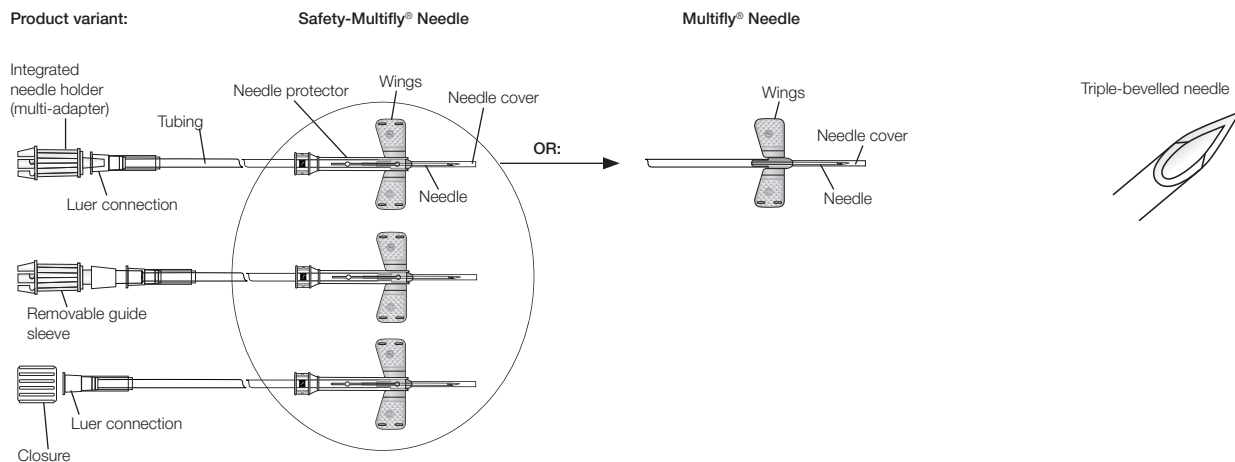
Intended use

The Safety-Multifly® Needle / Multifly® Needle is a sterile butterfly needle developed for single use. The Safety-Multifly® Needle / Multifly® Needle, together with a SARSTEDT S-Monovette®, is a system for venous blood collection. Additionally, the Safety-Multifly® Needle / Multifly® Needle can be used for a short-term infusion. The needle protector of the Safety-Multifly® Needle is used immediately after blood collection of the last S-Monovette® / short-term infusion to cover the used needle and thus reduce potential accidental needle stick injuries. The products are to be used by trained personnel.

Product description

The Safety-Multifly® Needle / Multifly® Needle is a sterile, pyrogen-free disposable product packed individually in a blister pack. Safety-Multifly® Needles / Multifly® Needles consist of a medical 3/4 inch (19 mm) long disposable stainless steel needle with a triple bevel, flexible tubing (80 mm or 200 mm) and an integrated needle holder (also called multi-adapter). There is a version of the Safety-Multifly® Needles / Multifly® Needles without an integrated needle holder (multi-adapter) with a tubing length of 80 mm or 240 mm. The products are available with the needle diameters 20, 21, 23, and 25 gauge (G). The needle protector of the Safety-Multifly® Needles is used to cover the needle after use. The products contain only components that are DEHP and latex free.

Product variant:

Safety information and warnings

1. Check all individual packaging for damage prior to use and, if damaged, do not use. Likewise do not use the product if the needle is bent or broken.
2. Follow the directions for handling.
3. Wear gloves and general personal protective equipment to protect yourself from blood and possible exposure to pathogens transmitted by blood.
4. Do not bend the needle.
5. Do not cover the safety mechanism of the Safety-Multifly® Needle with tape.
6. During use and disposal, always keep your hands behind the needle.
7. Do not use force to pull the butterfly needle or prematurely activate the safety mechanism of the Safety-Multifly® Needle (needle protector), as this would impair the functioning of the Safety-Multifly® Needle.
8. Injections and subcutaneous use are not allowed.
9. Before using for a short-term infusion, ensure that all the air is first removed (priming).
10. After activating the safety mechanism of the Safety-Multifly® Needle, do not open it again. Do not use force.
11. When removing or disposing of the needle, check for possible leakage of blood or medication. It is then recommended to dispose of the Safety-Multifly® Needle / Multifly® Needle with the tip of the needle pointing up with the needle protector activated (Safety-Multifly® Needle).
12. Dispose of used, contaminated Safety-Multifly® Needles / Multifly® Needles immediately in suitable sharps disposal containers and dispose of equipment for blood collection (e.g. lancets, adapters) in sharps disposal containers as well.
13. Treat all biological specimens and sharp blood collection utensils (needles) according to the guidelines and procedures in your institution. In case of exposure to biological specimens or a needle stick injury, consult a doctor, as HBV, HCV, or HIV or other infectious diseases can be transmitted. Use needles with an integrated needle protector. The safety guidelines and procedures of your institution must always be followed.
14. Reusing this product could result in infections, injury or death.

Contraindications

1. Do not give a short-term infusion or perform venous blood collection on patients with infections, burns, injuries or who have had vascular surgery at the planned puncture site.
2. Do not infuse highly venotoxic drugs through the butterfly needle.
3. Do not infuse drugs that have a strong effect on the circulatory system or drugs requiring infusion rate control through the butterfly needle.

General residual risks of venous blood collection and short-term infusion

When applying the basic technique for blood collection and short-term infusion, there are the following general residual risks in accordance with *CLSI GP41* and the *WHO guidelines on drawing blood*:

1. Pain
2. Bruising
3. Vasovagal reactions
4. Infections
5. Nerve injuries
6. Allergic reactions, mainly caused by disinfectants.

Collecting and handling specimens

READ THIS DOCUMENT IN FULL BEFORE PROCEEDING WITH VENIPUNCTURE.

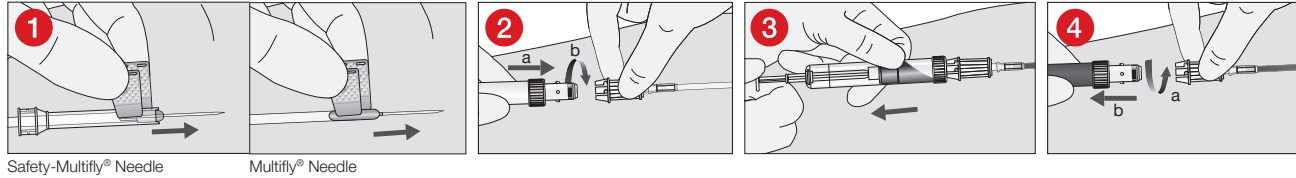
EQUIPMENT REQUIRED FOR SPECIMEN COLLECTION

1. All necessary S-Monovette® blood collection systems.
2. Safety-Multifly® Needles (for Safety-Multifly® Needles with Luer adapter, an additional multi-adapter (14.1205/14.1205.100) or male Luer adapter is needed for the blood collection)
3. Gloves, gown, goggles or other suitable protective gear to protect against pathogens or potentially infectious materials transmitted by blood.
4. Labels for patient identification.
5. Disinfectant to clean the collection site (follow the guidelines of the facility for aseptic specimen collection for preparing the collection site). Do not use any alcohol-based cleaning materials if the specimens are to be used for a blood alcohol test.
6. Dry, clean, disposable swabs.
7. A tourniquet.
8. Plaster or dressing.
9. Sharps disposal container for the safe disposal of used material.

Venous puncture with the Safety-Multify® Needle / Multify® Needle:

Apply tourniquet. Prepare the puncture site with a suitable disinfectant. Do not touch the venipuncture site after disinfection. Position the patient to allow easy access to the vein.


- 1 Open the blister packaging of the Safety-Multify® Needle / Multify® Needle aseptically using the peel-off technique with thumb and index finger. The transparent film must be facing upwards. After the disinfectant has dried, remove the protective sleeve from the needle and ensure that the bevelled side of the Safety-Multify® Needle/Multify® Needle is facing upwards. Perform the venous puncture while holding the coloured wings together upwards.
- 2 Slide the S-Monovette® into the multi-adapter of the Safety-Multify® Needle / Multify® Needle (a) and secure (connect) it with a ¼ turn clockwise (b).
- 3 Draw blood with the S-Monovette® by carefully withdrawing the plunger up to the bottom of the S-Monovette®. Loosen the tourniquet when collecting blood with the first S-Monovette®.
- 4 After completing blood collection, first twist the S-Monovette® ¼ turn counterclockwise (a) and then pull it out of the Safety-Multify® Needle / Multify® Needle (disconnect) (b). The Safety-Multify® Needle / Multify® Needle remains in the vein while blood is collected and when the blood collection tubes are changed. For multiple specimens – as described above – the subsequent S-Monovettes are connected to the Safety-Multify® Needle / Multify® Needle and additional specimens are taken. At the end of blood collection, the last S-Monovette® is disconnected from the Safety-Multify® Needle / Multify® Needle and step A or step B are followed for the Safety-Multify® Needle and step C for the Multify® Needle.

For the Safety-Multify® Needle:


- A. The Safety-Multify® Needle is withdrawn from the vein (5) and the needle protector is activated outside the vein (6) by pushing the needle protector over the needle with the thumb and index finger, while holding the tubing in place with the other fingers. The needle must perceptibly engage in the needle guard with an audible "click" (6).
- OR:
- B. The needle guard is activated in parallel with withdrawing the Safety-Multify® Needle from the vein (5). The needle must perceptibly engage in the needle guard with an audible "click" (6).

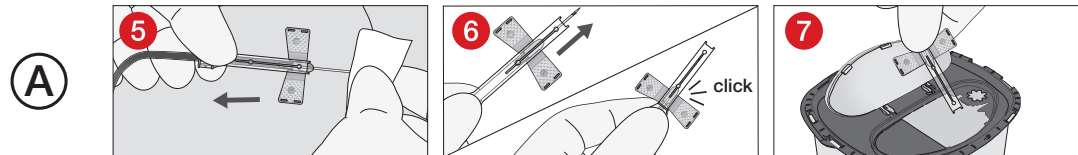The safety mechanism is designed to be activated using a one-handed technique.

Activating the needle protector – needle outside the vein: Withdraw the Safety-Multify® Needle from the vein and activate the needle protector.

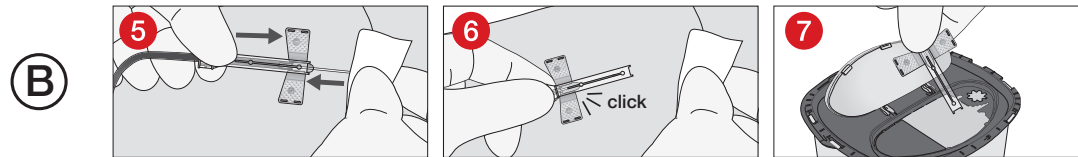
OR:

Activating the needle protector – needle in the vein: Activate the needle protector in parallel with withdrawing the Safety-Multify® Needle from the vein.

For the Multify® Needle:

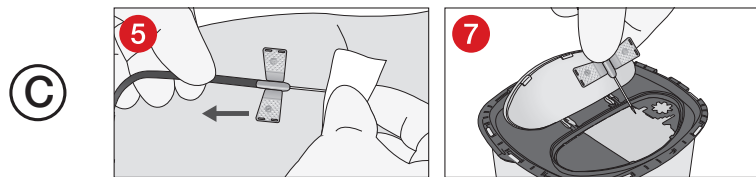- C. The Multify® Needle is withdrawn from the vein (5).

Dispose of the Multify® Needle (7) and the safely locked Safety-Multify® Needle (7, illustration of Safety-Multify® Needle) immediately in a container for sharp objects.

Handling for blood collection: see our handling video for the Safety-Multify® Needle: [www.sarstedt.com/en/download/videos/ Diagnostics / Venous blood](http://www.sarstedt.com/en/download/videos/Diagnostics/Venous%20blood)

Short-term infusion with the Safety-Multify® Needle / Multify® Needle

Prepare the puncture site with a suitable disinfectant. Do not touch the venipuncture site after disinfection. Position the patient to allow easy access to the vein.

- 8 Open the blister packaging of the Safety-Multify® Needle / Multify® Needle aseptically using the peel-off technique with thumb and index finger. The transparent film must be facing upwards. Remove the multi-adapter at the end of the hose by twisting and pulling it. After the disinfectant has dried, remove the Luer lock of the infusion system and connect the female Luer connection at the end of the Safety-Multify® / Multify® needle hose with male Luer connection of the infusion system.
- 9 Prefill the infusion system according to the guidelines of the institution.




Maximum venting volume of the Safety-Multify® Needles / Multify® Needles with needle diameters 20G, 21G, 23G, 25G:

240 mm hose:	0.450 ml
200 mm hose:	0.300 ml
80 mm hose:	0.120 ml

Remove the protective sleeve from the needle and ensure that the bevelled side of the Safety-Multify® Needle/ Multify® Needle is facing upwards. Perform the venipuncture while holding the coloured wings (1) together upwards. Then a short infusion (max 24h) can be carried out.

10 To end the short infusion, the infusion system is disconnected from the Safety-Multify® Needle / Multify® Needle and the Safety-Multify® Needle/ Multify® Needle is then pulled out of the vein by following step A or step B for the Safety-Multify® Needle and step C for the Multify® Needle.

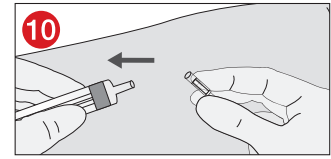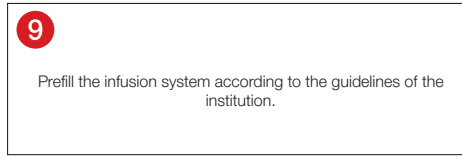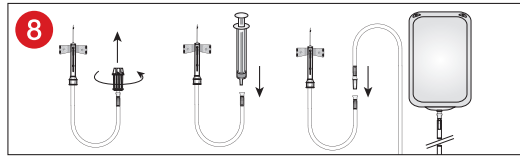Dispose of the Multify® Needle (7, illustration of Multify® Needle) and the safety locked Safety-Multify® Needle (7, illustration of Safety-Multify® Needle) immediately in a container for sharp objects (7).

Storage

Store the Safety-Multify® Needle / Multify® Needle at room temperature.

NOTE: Do not expose to direct sunlight. Exceeding the maximum recommended storage temperature may affect the quality of the Safety-Multify® Needle / Multify® Needle.

Product-specific standards and guidelines as amended / updated

CLSI* GP41 Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture.

*CLSI (Clinical and Laboratory Standards Institute)

WHO guidelines on drawing blood: best practice in phlebotomy

DIN 13097-4: Hypodermic needles - Part 4: Point geometry, requirements and testing.

DIN EN ISO 6009: Hypodermic needles for single use - Colour coding for identification.

DIN EN 10088-1: Stainless steels - Part 1: List of stainless steels.

DIN EN ISO 9626: Stainless steel needle tubing for the manufacture of medical devices - Requirements and test methods.

Key for symbols and labels:

	Medical device		Follow the instructions for use
	Order no.		Country of manufacture
	Batch number		Keep away from sunlight
	CE mark		Store in a dry place
	Manufacturer		Do not use if package is damaged
	Store at room temperature		non-pyrogenic
	Sterilisation with ethylene oxide gas		DEHP free
	Risk of contamination if reused		Single sterile barrier system
	Do not resterilise		Single sterile barrier system with protective packaging outside
	Use by		

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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